

LION™

First of its Kind Fully Integrated LIO and Green Laser



The laser source is smaller than this coin.

NO FIBER TETHER = NO COSTLY REPAIRS



LION Green Laser

Untethered design for anywhere, anytime treatment convenience.



Superior Visualization

Ultra-clear optics enhance visualization during patient diagnosis and treatment.

Touchscreen Tablet and Control Panel

Control everything through a user-friendly interface on your sleek wireless tablet.

Activate optional Voice Recognition Mode for touchless parameter control and stay focused on your patients.



Footswitch with Rechargeable Battery

A single charge will give you days of laser use.



Start using the LION Laser in a few simple steps!

LION™ | QUICK START INSTRUCTIONS



IMPORTANT: Familiarity with indirect ophthalmoscope use is required for this quick start guide.

1 CONNECTING LION



Connect the LION headset cable to the back of the footswitch mating cable receptacle. Please note: this connector is keyed, so it must be inserted in the proper orientation with the arrow on the connector facing up.

2 TURN ON LION



Locate the key switch on right side of the LION. Insert key into key switch and turn on. The light on the back of the LION headset will blink blue as it is establishing connection to the tablet.

3 TURN ON TABLET



Place the tablet on the stand and press the power button to turn on the tablet. The light in the back of the LION headset will turn green when successfully connected.

4 SELECT PARAMETERS



Select the desired LION headset starting parameters by using the touchscreen interface.

5 MOUNT ON HEAD



Mount the LION headset on your head and adjust for clear viewing through oculars.

6 SELECT READY MODE



Select READY mode. The aiming beam will turn on.* Your LION headset is now ready to use.



For a complete library of video instructions, please scan the QR code or visit www.norlase.com/lion-ifu

*All ancillary persons in the treatment room or present during time of laser treatment are required to wear laser safety glasses approved for use with the LION green laser. The quick start instructions together with the collection of instructional videos are intended to be supplemental resources for your convenience. Please refer to the LION Operator Manual before using the LION Green laser for important safety and product-use instructions.

FOR QUESTIONS AND INQUIRIES:

Web: www.norlase.com | Email: contact@norlase.com | USA: 1.650.489.0083 | Denmark: + 45 9397 7472